

# Information sheet PTS

## Swine influenza virus detection – VLDIA347

These products are produced by the Royal GD and meant to be used in the Proficiency testing scheme.  
(spiked medium on FTA cards – Swine Influenza virus, SIV)

### Samples

The sample set consists of 8 coded samples. The samples are spiked medium containing Swine influenza virus. The samples have to be tested in duplicate (in different test runs) using the method(s) (of choice) of your laboratory. The samples should be tested under normal routine test conditions.

- 1 Set PTS samples.  
1 set consists of 8 coded samples (spiked medium on FTA Cards) containing Swine influenza virus.
- The samples should be stored at 2-8 °C and tested within 5 days.
- Sample will be sufficient to perform the required tests.

These products are produced, bottled and sealed at Royal GD and meant to be used in the Proficiency testing scheme.

No commercial value

All samples are chemically inactivated.

### Reporting your findings

You can use Nucleic acid Amplification Techniques (NAT), e.g. PCR.

Results can be reported as positive-negative and/or Ct values.